

Instruction for Use

Trade Name: Four-point spinal frame

Warning


1. Please read these instructions carefully prior to using this product. Product should be used according to these instructions and pay close attention to the safety of patients. Not doing so may give rise to serious problems or adverse events..
2. When the device is used on a patient where there is a suspicion of Creutzfeldt-Jakob Disease (CJD), make sure to adhere to the most recent and updated restrictions available in each country and/or region for its reuse. [Be cautious of the possibility of secondary infection.]

Contraindication/Prohibition

1. Use for intended purpose only
Use tools for their intended purposes only. Incorrect use could cause breakage of this product.
2. Prohibition of use of chemicals
Avoid exposing this product to chemicals. Doing so could damage this product due to corrosion.
3. Prohibition of secondary processing of this product
Do not apply any secondary processing to this product. For example, do not apply impacts or vibration markings to the surface of this product. Doing so could break this product.
4. Handle with care
Handle this product with care, as it can be deformed or damaged. Rough handling could significantly reduce the service life of tools and appliances.
5. Prohibition of use of polishing powder and wire wool
When cleaning this product, do not attempt to polish its surfaces with rough polishing powder or wire wool. This could cause scratches on the surfaces of this product and result in rust or corrosion.
6. Prohibition of sterilization
Do not sterilize this product. Sterilization could affect the properties of this product.



Shape/ Structure

1. Shape/Structure

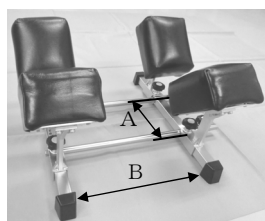

Code No.	Product Description
02-121-00	Four-point spinal frame

2. Material : Stainless steel, Plastic, Cloth material
3. Weight :9.7 kg
4. Adjustable range: Direction A 100 mm, Direction B 243 mm

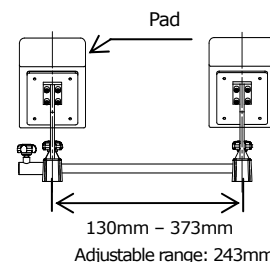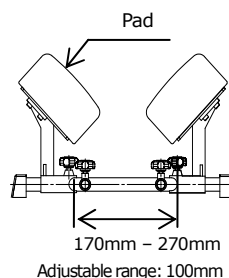

Intended purpose

This is an accessory to an operating table, and is used for supporting a patient to keep the prone position.

Directions for Use

1. Preparations before use
Before using this product, be sure to inspect, wash it under proper conditions and disinfect.
2. Instructions for use
 - 2-1 Make sure the table is locked after the brake is applied.
 - 2-2 Make sure that the table top is in the level position.
 - 2-3 Attach this product to the tabletop.
 - 2-4 Move the pad to a position appropriate for the patient's posture.

Precautions before use

1. Prohibition of sterilization
Do not sterilize this product. Sterilization could affect the properties of this product
2. A fault or an adverse event
-Prohibition of use of chemicals
Avoid exposing this product to chemicals. Doing so could damage this product due to corrosion.
3. Allowable Load
Do not add load more than 60kg.

Storage/Life

1. Please store the device in normal ambient temperature areas. Do not store in areas of high humidity where the temperature may vary wildly causing condensation. Do not store on or near chemicals.
2. Service life of this product: 7 years
(Subject to the specified maintenance, inspection, and proper storage)

Maintenance/Inspection

<By the user>

1. Check before and after use
Check that this product is not damaged or broken before and after use.
2. Cleaning and disinfection
Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other material moistened with alcohol to disinfect.

Cleaning	Disinfection*1	Sterilization		
		EOG	high-presure steam	hydrogen peroxide gas plasma
Wipe with medical neutral detergents	Clean with alcohol	OK	NG	NG
OK	OK	NG	NG	NG

*1: Iodine-based disinfectant, is not available for any parts

3. In case of a malfunction

If you need to have this product repaired, clearly label the device as "Broken", "Need repair" etc. and contact your local dealer or Mizuho.

<By suppliers>

Maintenance and inspection can only be carried out certified suppliers of Mizuho.

Packing

1 set per package

Matters related to warranty period

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or damage on purpose. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

***Name and address of manufacturer**



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<http://www.mizuho.co.jp>

*** Authorized Representative Europe**



Emergo Europe B.V.
Prinsessegracht 20, 2514 AP, The Hague
The Netherlands