

## Instruction for Use

Trade Name: Horseshoe Head Rest

### Warning

When using operating table accessories, always pay close attention to the patient. Patients left in the same posture for an extended period of time are at risk for developing nerve paralysis and/or bedsores.

### Contraindication/Prohibition


1. This product is only validated for use with MIZUHO operating tables.
2. Use this product only with products specified by Manufacturer. Other products than those specified by Manufacturer could be incompatible with this product due to differences in design and development policies. Refer to **Instructions for use** 2. Applicable accessories.

### Symbol mark for labeling

**MD** : Medical Device

### Specifications

#### 1. Shape

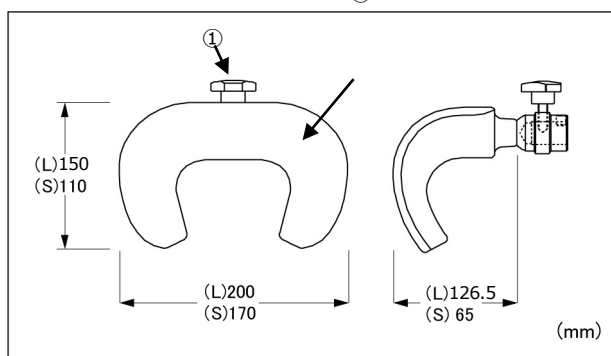

Code No.	Product Description	Component
07-953-03	Horseshoe Head Rest, L	①②
07-953-04	Horseshoe Head Rest, S	①②

#### 2. Material

- ① Horseshoe Head Rest: Polyurethane
- ② Fixing handle: Stainless steel

Note: The number corresponds to those used in 1. Shape.

### Intended Purpose


This is an accessory for the operating table. This product is used for supporting the patient's head.

### Intended user

This product is to be used by health care professionals, including but not limited to surgeons, nurses and biomedical technicians.

### Instructions for use

#### 1. Instructions for use

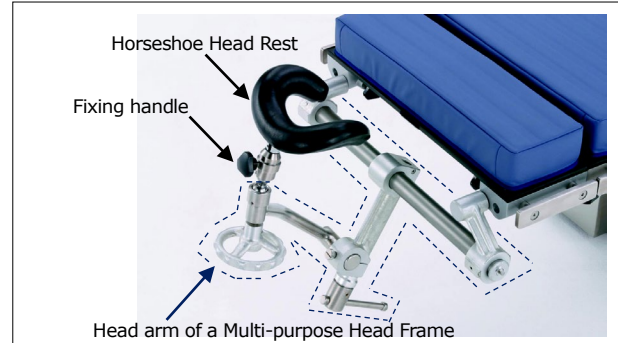

1. Attach the Horseshoe Head Rest to the head arm of a Multi-purpose Head Frame or Sitting Position Frame with Clamps, and adjust the Horseshoe Head Rest to the desired position.  
(Above figure: attached to the head arm of a Multi-purpose Head Frame)
2. Tighten the fixing handle to fix the Horseshoe Head Rest.

#### 2. Applicable accessories

Code No.	Product Description
18-952-00	Table Attachment
18-953-01	Sitting Position Frame with Side Rail Clamp

### Warning/Caution

#### Important caution

1. Ensure all fixing handles are tight. Loose condition can cause the product to come off or move, which may result in a patient injury.
2. When working with the operating table, take care not to allow this product to make contact or interfere with the tabletop or with other tools and appliances used in combination with the product. Otherwise the product could break.
3. For hygiene, be sure to use sterilized drapes on the areas on this product where the patient comes into contact with it.

### Storage/Life

1. Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
2. Service life of this product: 7 years  
(Subject to the specified maintenance and inspection and is stored properly.)

### Maintenance/Inspection

#### <By the user>

1. Check before and after use  
Check that this product is not damaged or broken before and after each use.
2. Cleaning and disinfection  
Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other materials soaked in disinfectants. In addition, use the same disinfectants as the authorized disinfectants on MIZUHO's operating table described in the operating table operator's manual.
3. In case of malfunction  
When this product is broken, clearly label the device as "Broken", "Do not use", "Need repair", etc., and contact your local distributor or MIZUHO.

#### <By agents>

Maintenance and inspection can only be carried out by certified agents of MIZUHO.

**Packing**

1 piece per package

**Warranty**

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

**Disposal**

This device must be disposed of in accordance with local regulations. Please contact your local distributor for proper disposal.

**Notice**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.



MIZUHO Corporation  
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan  
<https://www.mizuho.co.jp>



EMERGO EUROPE  
Westervoortsedijk 60  
6827 AT Arnhem, The Netherlands