

Be sure to read this instruction manual before using this product.
 Keep this manual available for reference when needed.



Instruction for Use

Trade Name: Back Section Board Short

Warning
 When using operating table accessories, always pay close attention to the patient. Patients left in the same posture for an extended period of time are at risk for developing nerve paralysis and/or bedsores.




Contraindication/Prohibition
 1. This product is only validated for use with Mizuho operating tables.
 2. Use only on the models listed in **Instructions for use**, 2.Applicable operating tables.

Symbol mark for labeling


MD : Medical Device

Specifications

1. Shape

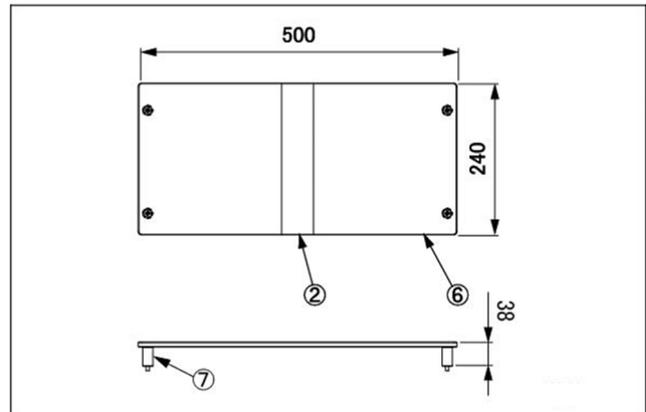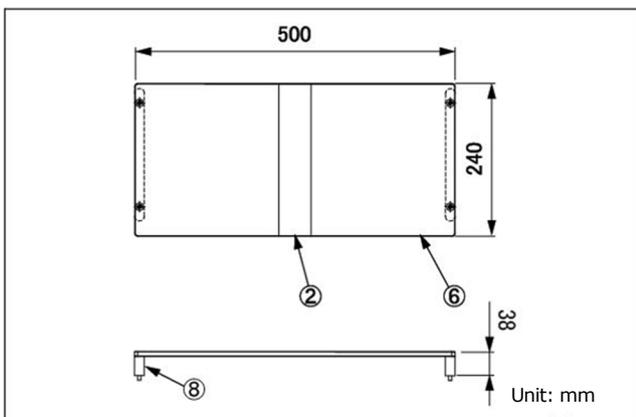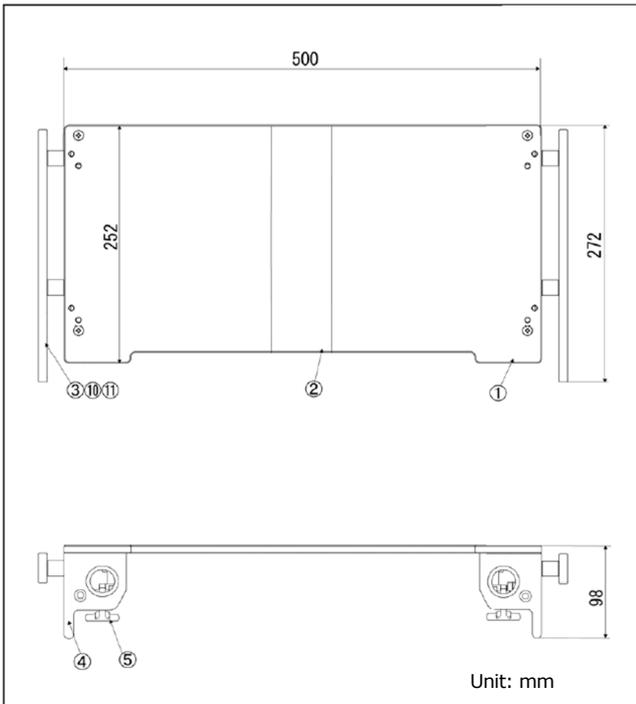

Unit: mm

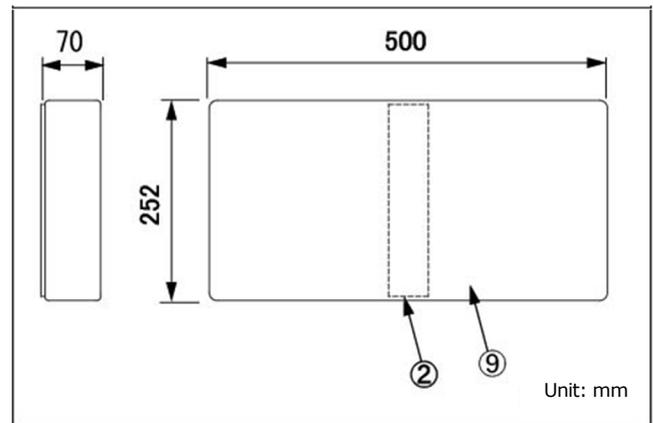

Code No.	Product Description	Compo nents	Weight
08-007-21	Back Section Board Short *2	①②③ ④⑤	6.0 kg
08-007-21-R1	Back Section Board Short (R1) *3	①②④ ⑤⑩	6.0 kg
08-007-21-R2	Back Section Board Short (R2) *4	①②④ ⑤⑪	6.0 kg
08-083-43	Phenolic Board for Radiographic and Digital Radiographic Table Tops Fixed Type for Back Section Board Short	①	1.4 kg
08-083-23	Digital Radiographic Table Tops Fixed Type for Back Section Board Short	②⑥⑧	1.4 kg
08-083-03	Radiographic Table Tops Fixed Type for Back Section Board Short	②⑥⑦	1.4 kg
08-083-64	Mizuho Original Pad (252mm)	②⑨	0.5 kg
18-042-80	SUSANO Pad for Back Section Board Short	②⑨	0.5 kg

*1: The number corresponds to those used in 1. Shape.
 *2,*3 and *4: The size of mountable side rail is different.

2. Weight

Refer to the table shown above.

3. Material

- ① Board: Thermosetting resin
- ② Velcro tape: Nylon
- ③ Side rail: Stainless steel
- ④ Frame: Aluminum alloy
- ⑤ Locking levers: Stainless steel
- ⑥ Radiographic top: Thermosetting resin
- ⑦ Cassette collar: Stainless steel
- ⑧ Radiographic top leg: Polycarbonate
- ⑨ Pad: Polyurethane
- ⑩ Side rail (R1) : Stainless steel
- ⑪ Side rail (R2) : Stainless steel

Note: The number corresponds to those used in 1. Shape.

Intended purpose

This is an accessory for the operating table. This product is intended to be attached to the table for supporting a patient's back. It can be used for X-ray imaging.


Intended user

This product is to be used by health care professionals, including but not limited to surgeons, nurses and biomedical technicians.

Instructions for use

1. Attachment

- 1-1. Move the back section of the operating table to horizontal, and detach the head section and the back section.

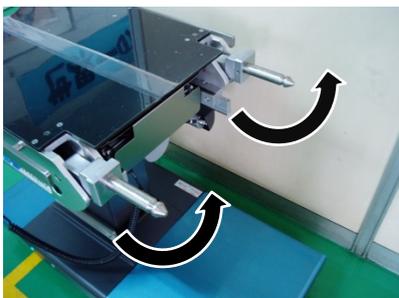

Note: State whereby the head section and the back section are detached.

- 1-2. Check that the pins of the operating table back section are in a horizontal position, and insert the Back Section Board Short.



Note: State whereby the Back Section Board Short is attached.

- 1-3. Pull the Back Section Board Short toward you, and check that it is fully attached to the operating table.

Note: The photo shows the Back Section Board Short.

2. Applicable operating tables

Refer to the brochure "OPERATING TABLE ACCESSORIES" of Mizuho.

Warning/ Caution

1. Warning

Ensure all attaching parts are fixed. Loose condition can cause the product to come off or move, which may result in a patient injury.

2. Important caution

2-1. Make sure that the Back Section Board Short is fully inserted into the pins of the operating table.

2-2. When working with an operating table, take care not to allow this product to make contact or interfere with the tabletop or with other tools and appliances used in combination with this product. Otherwise the product could break.

2-3. For hygiene, be sure to use sterilized drapes on the areas on this product where the patient comes into contact with it.

Storage/Life

1. Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.

2. Service life of this product: 7 years

(Subject to the specified maintenance and inspection and is stored properly.)

Maintenance/Inspection

<By the user>

1. Check before and after use

Check that this product is not damaged or broken before and after each use.

2. Cleaning and disinfection

Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other materials soaked in disinfectants. In addition, use the same disinfectants as the authorized disinfectants on Mizuho's operating table described in the operating table operator's manual.

3. In case of malfunction

When this product is broken, clearly label the device as "Broken", "Do not use", "Need repair", etc., and contact your local dealer or Mizuho.

<By agents>

Maintenance and inspection can only be carried out by certified agents of Mizuho.

Packing

1 piece per pack

Warranty

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

Disposal

This device must be disposed of in accordance with local regulations. Please contact your local distributor for proper disposal.

Notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<https://www.mizuho.co.jp>



EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem, The Netherlands