

## Instruction for Use

Trade Name: Uro Drain Tray Set

### Contraindication/Prohibition






Use only on Universal Operating Table SPL-331 of Mizuho.

### Symbol mark for labeling

**MD** : Medical Device

### Specifications

#### 1. Shape

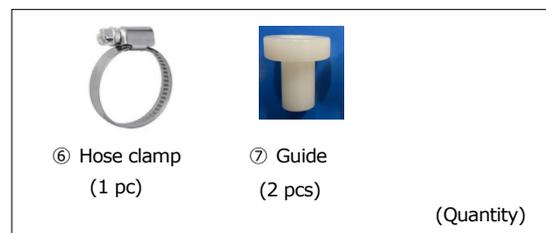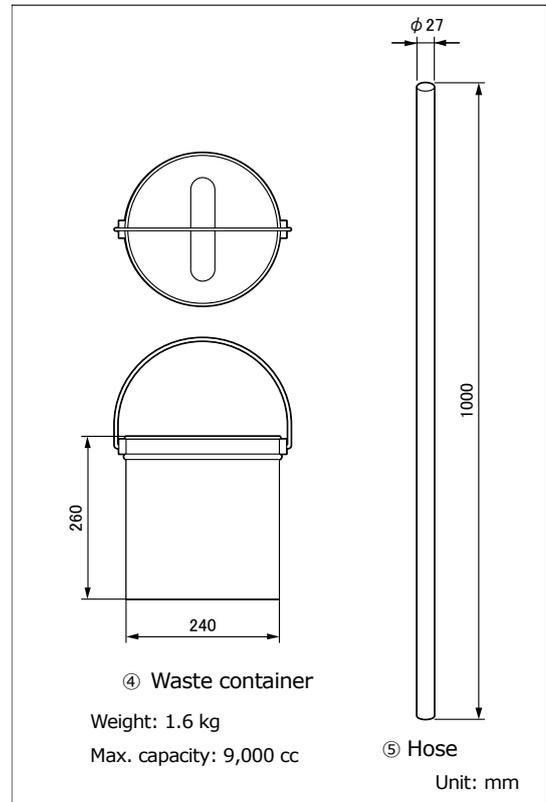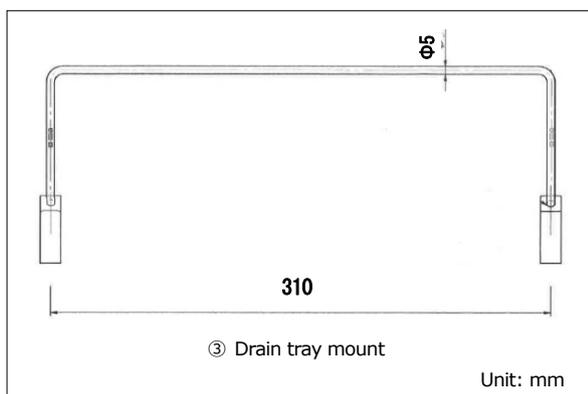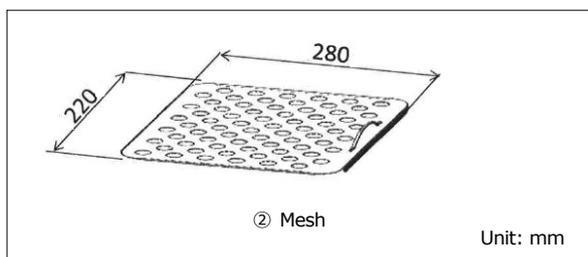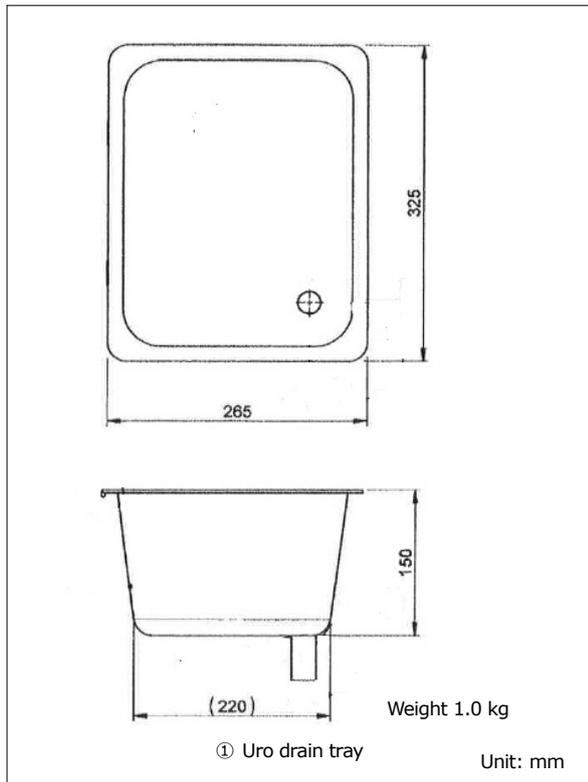

Code No.	Product Description	Components
08-010-11	Uro Drain Tray Set for SPL-331	①②③④⑤⑥⑦
08-095-19	Uro Drain Tray for SPL-331	①②③⑥⑦
08-095-20	Waste Container with Hose	④⑤

#### 2. Material

- ① Uro drain tray: Stainless steel
- ② Mesh: Stainless steel
- ③ Drain tray mount : Stainless steel
- ④ Waste container: Stainless steel
- ⑤ Hose: Nylon
- ⑥ Hose clamp: Stainless steel
- ⑦ Guide: Nylon

Note: The numbers correspond to those used in 1. Shape.

### Intended purpose

This is an accessory for the operating table. This product is used for the surgical washing in the urology, gynecology, etc. A fine mesh sieve is used in the drain tray in order to prevent a splash of washing liquid and trap foreign matters. The uro drain tray is to be attached to guides secured to the operating table with provided screws.


### Intended user

This product is to be used by health care professionals, including but not limited to surgeons, nurses and biomedical technicians.


### Instructions for use

#### 1. Attachment

1-1. Attach the drain tray mount<sup>③</sup> to the operating table with screws.

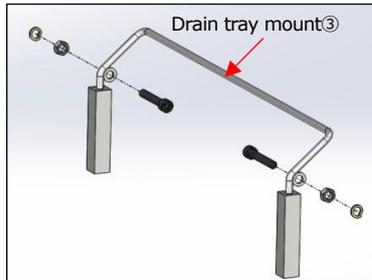

1-2. Attach the guides<sup>⑦</sup>.

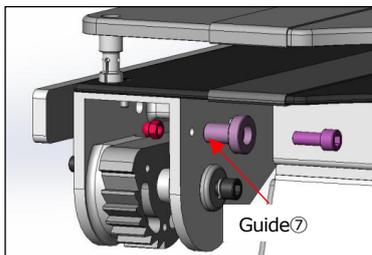


1-3. Insert the uro drain tray<sup>①</sup> into the guides<sup>⑦</sup>.




1-4. Pull the uro drain tray<sup>①</sup> toward you to check that the stopper works to prevent the uro drain tray<sup>①</sup> from coming off.



1-5. Attach the mesh<sup>②</sup>.

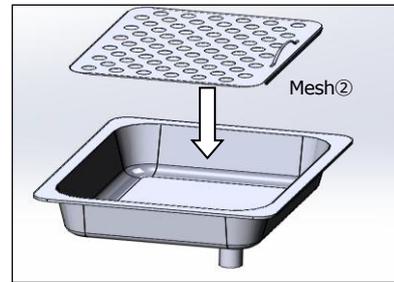

1-6. Attach the hose<sup>⑤</sup> and the waste container<sup>④</sup> to the uro drain tray<sup>①</sup> with the hose clamp<sup>⑥</sup>.

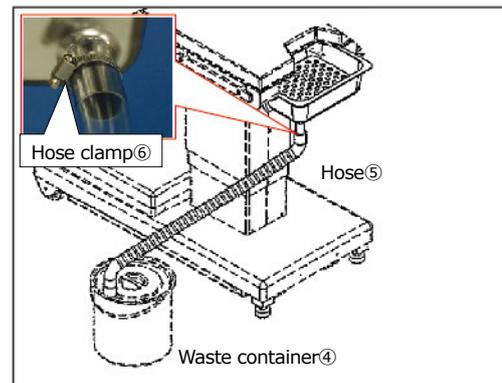

Note: The numbers correspond to those used in Specifications, 1. Shape.

#### 2. Detachment

2-1. Detach the mesh<sup>②</sup>, the hose<sup>⑤</sup>, and the waste container<sup>④</sup> from the uro drain tray<sup>①</sup>.

2-2. While pulling the drain tray mount<sup>③</sup> upward, pull the uro drain tray<sup>①</sup> toward you.



Note: The numbers correspond to those used in Specifications 1. Shape.

### Warning/Caution

#### 1. Warning

Ensure all fixing screws are tight. Loose condition can cause the product to come off or move, which may result in a patient injury.

#### 2. Important caution

When working with an operating table, take care not to allow this product to make contact or interfere with the table top or with other tools and appliances used in combination with this product. Otherwise the product could break.

### Storage/Life

1. Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
2. Service life of this product: 7 years  
(Subject to the specified maintenance and inspection and is stored properly.)

## Maintenance/Inspection

<By the user>

### 1. Check before and after use

Check that this product is not damaged or broken before and after each use.

### 2. Cleaning and disinfection

Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other materials soaked in disinfectants. In addition, use the same disinfectants as the authorized disinfectants on Mizuho's operating table described in the operating table operator's manual.

### 3. In case of a malfunction

When this product is broken, clearly label the device as "Broken", "Do not use", "Need repair", etc., and contact your local distributor or Mizuho.

<By agents>

Maintenance and inspection can only be carried out by certified agents of Mizuho.

## Packing

1 set per pack

## Warranty

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

## Disposal

This device must be disposed of in accordance with local regulations. Please contact your local distributor for proper disposal.

## Notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.



MIZUHO Corporation  
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan  
<https://www.mizuho.co.jp>



EMERGO EUROPE  
Westervoortsedijk 60  
6827 AT Arnhem, The Netherlands