

Instruction for Use

Trade Name: U-Shaped Head Rest

Warning

When using operating table accessories, always pay close attention to the patient. Patients left in the same posture for an extended period of time are at risk for developing nerve paralysis and/or bedsores.

Contraindication/Prohibition


1. This product is only validated for use with MIZUHO operating tables.
2. Use this product only with products specified by Manufacturer. Other products than those specified by Manufacturer could be incompatible with this product due to differences in design and development policies. Refer to **Instructions for use 2. Applicable accessories.**

Symbol mark for labeling

MD : Medical Device

Specifications

1. Shape

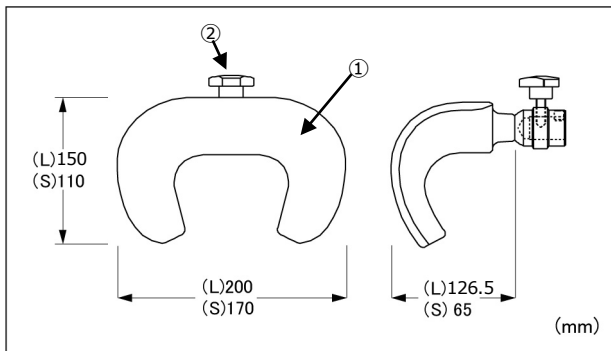

Code No.	Product Description	Component
08-073-03	U-Shaped Head Rest L	①②
08-073-04	U-Shaped Head Rest S	①②

2. Material

- ① U-Shaped Head Rest: Polyurethane
- ② Fixing handle: Stainless steel

Note: The number corresponds to those used in 1. Shape.

Intended Purpose


This is an accessory for the operating table. This product is used for supporting the patient's head.

Intended user

This product is to be used by health care professionals, including but not limited to surgeons, nurses and biomedical technicians.

Instructions for use

1. Instructions for use

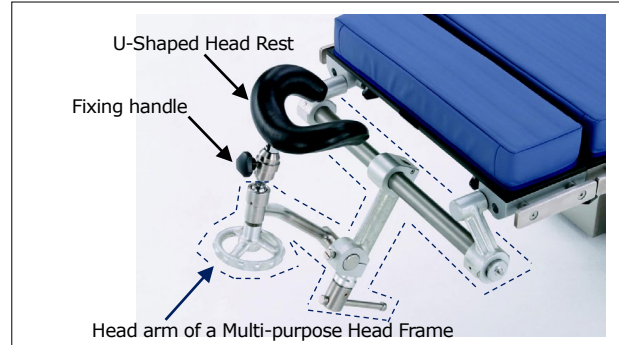

1. Attach the U-Shaped Head Rest to the head arm of a Multi-purpose Head Frame or Sitting Position Frame with Clamps, and adjust the U-Shaped Head Rest to the desired position.
(Above figure: attached to the head arm of a Multi-purpose Head Frame)
2. Tighten the fixing handle to fix the U-Shaped Head Rest.

2. Applicable accessories

Code No.	Product Description
07-952-00	Table Attachment
07-953-01	Sitting Position Frame with Side Rail Clamp

Warning/Caution

Important caution

1. Ensure all fixing handles are tight. Loose condition can cause the product to come off or move, which may result in a patient injury.
2. When working with the operating table, take care not to allow this product to make contact or interfere with the tabletop or with other tools and appliances used in combination with the product. Otherwise the product could break.
3. For hygiene, be sure to use sterilized drapes on the areas on this product where the patient comes into contact with it.

Storage/Life

1. Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
2. Service life of this product: 7 years
(Subject to the specified maintenance and inspection and is stored properly.)

Maintenance/Inspection

<By the user>

1. Check before and after use
Check that this product is not damaged or broken before and after each use.
2. Cleaning and disinfection
Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other materials soaked in disinfectants. In addition, use the same disinfectants as the authorized disinfectants on MIZUHO's operating table described in the operating table operator's manual.
3. In case of malfunction
When this product is broken, clearly label the device as "Broken", "Do not use", "Need repair", etc., and contact your local distributor or MIZUHO.

<By agents>

Maintenance and inspection can only be carried out by certified agents of MIZUHO.

Packing

1 piece per package

Warranty

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

Disposal

This device must be disposed of in accordance with local regulations. Please contact your local distributor for proper disposal.

Notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<https://www.mizuho.co.jp>



EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem, The Netherlands