

## Instruction for Use

### Trade Name: Accessory Cart


#### Contraindication/Prohibition

Do not use it for any purpose other than for storing and transporting the operating table accessories. Misuse could cause injury or damage.

#### Specifications

1. Shape

Unit: mm

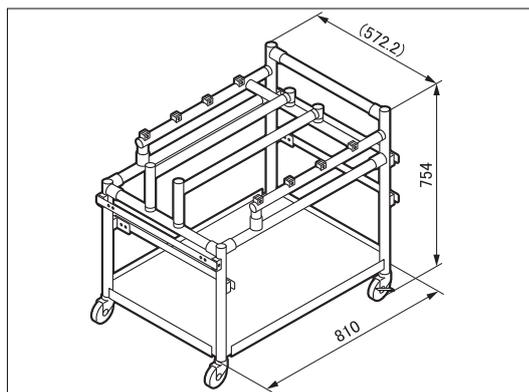

Code No.	Product Description
08-081-09	Accessory Cart (B)
08-081-09-R1	Accessory Cart (B) (R1)
08-081-09-R2	Accessory Cart (B) (R2)

Note: The size of mountable side rail is different.

2. Weight

15 kg

3. Material

Main unit: Stainless steel, aluminum alloy, rolled steel

Caster: Synthetic resin

#### Intended purpose


This is a cart for storing and transporting the operating table accessories.

#### Intended user

This product is to be used by health care professionals, including but not limited to surgeons, nurses and biomedical technicians.

#### Instructions for use

Store and transport the accessories as shown in the figure.

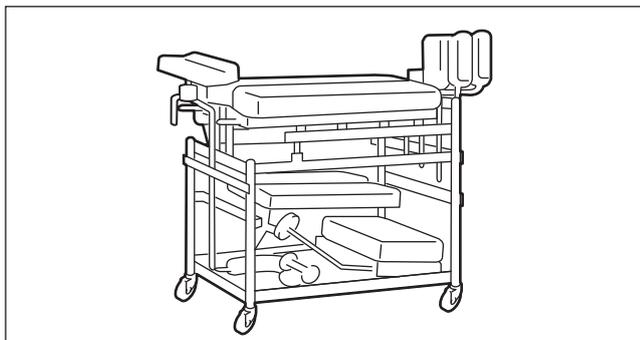

Note: The figure shows an example.

#### Warning/Caution

Important caution

Do not move the cart when accessories are not stored appropriately. Failure to do so may result in the accessories falling down and a person injury.

#### Storage/Life

- Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
- Service life of this product: 7 years  
(Subject to the specified maintenance and inspection and is stored properly.)

#### Maintenance/Inspection

<By the user>

- Check before and after use  
Check that this product is not damaged or broken before and after each use.
- Cleaning and disinfection  
Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other materials soaked in disinfectants. In addition, use the same disinfectants as the authorized disinfectants on Mizuho's operating table described in the operating table operator's manual.
- In case of a malfunction  
When this product is broken, clearly label the device as "Broken", "Do not use", "Need repair", etc., and contact your local distributor or Mizuho.

<By agents>

Maintenance and inspection can only be carried out by certified agents of Mizuho.

#### Warranty

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

#### Packing

1 piece per pack

#### Disposal

This device must be disposed of in accordance with local regulations. Please contact your local distributor for proper disposal.

#### Notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.



MIZUHO Corporation  
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan  
<https://www.mizuho.co.jp>