

# Instruction for Use

Trade Name: Body Strap

### Warning

1. Please read these instructions carefully prior to using this product. Product should be used according to these instructions, and pay close attention to the safety of patients. Failure to follow manufacturer's recommendations may cause harm or injury to the patient.
2. For the US market  
Do not reuse the device when it is used on a patient with or suspected of having Creutzfeldt-Jakob Disease (CJD) or variant CJD (vCJD).
3. For the market outside the US  
When the device is used on a patient with or suspected of having CJD or vCJD, make sure to adhere to the most recent and updated restrictions available in each country and/or region for its reuse. Be cautious of the possibility of secondary infection. Refer to [www.a-k-i.org](http://www.a-k-i.org) or AAMI Standard ST79 for more information related to cleaning and sterilization.

### Contraindication/Prohibition

1. Use for intended purpose only  
Use devices for their intended purposes only. Incorrect use could cause this product to break.
2. Use with specified products only  
Use this product only with products specified by Manufacturer. Other products than those specified by Manufacturer could be incompatible with this product due to differences in design and development policies.
3. Prohibition of use of chemicals  
Avoid exposing this product to chemicals. Doing so could damage this product due to corrosion.
4. Prohibition of secondary processing of this product  
Do not apply any secondary processing to this product. For example, do not apply impacts or vibration markings to the surface of this product. Doing so could break this product.
5. Handle with care  
Handle this product with care, as it can be deformed or damaged. Rough handling could significantly reduce the service life of devices and appliances.

### Symbol mark for labeling

:Medical Device

### Specifications

Shape/Structure



Code No.	Product Description
08-084-01	Body Strap (2500 mm)

Material: Stainless steel, Vinyl leather, Urethane, Polyamide

### Intended purpose


This is an accessory to an operating table and is used to hold the torso of a patient.

### Intended user

This product is to be used by health care professionals, including but not limited to surgeons, nurses and biomedical technicians.

### Instructions for use

Be sure to inspect prior to using.

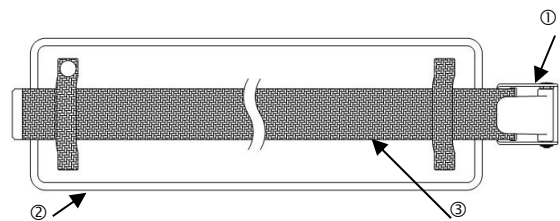

①Buckle      ②Pad      ③Strap

1. Attach the pad ② to the patient in a manner appropriate for his/her posture.
2. Wrap the strap ③ around the operating table.
3. Run the strap ③ through the buckle ① to secure the strap.

### Warning/Caution

(1) Defect/Adverse event

Defect

- Deterioration, corrosion or pitting caused by use of chemicals
- Damage or breakage caused by the corrosion or pitting

Adverse event

- Broken pieces of metal from the damaged instrument falling into the patient.

(2) For hygiene, be sure to use sterile drapes on the areas on this product where the patient comes into contact with it.

### Storage/Life

1. Please store the device in normal ambient temperature areas. Do not store in areas of high humidity where the temperature may dramatically vary causing condensation. Do not store on or near chemicals as the chemicals may cause damage to the device.
2. Service life of this product: This product is a consumable. Replace it with a new product as soon as possible if deterioration of the fastening force is observed.

### Maintenance/Inspection

Check prior to each use

1. Operational and functional checks  
Conduct daily and pre-operation checks of this product to make sure that it functions properly.
2. After use, wipe with a clean, non-fluffy cloth moistened with alcohol. If blood, etc., adheres, dispose of properly according to the laws and guidelines of each country or region.
3. If you need to have this product repaired, consult your local dealer or Mizuho.

### Packing

1 piece per pack

**Warranty**

Mizuho Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of Mizuho Corporation.

**Disposal**

This device must be disposed of in accordance with local regulations. Please contact your local distributor for proper disposal.

**Notice**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.



MIZUHO Corporation  
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan  
<https://www.mizuho.co.jp>



EMERGO EUROPE  
Westervoortsedijk 60, 6827 AT Arnhem  
The Netherlands