

Instruction for Use

Trade Name: Arm Strap

Warning

When using operating table accessories, always pay close attention to the patient. Patients left in the same posture for an extended period of time are at risk for developing nerve paralysis and/or bedsores.

Contraindication/Prohibition


Do not restrain the patient's arm forcibly. It may result in a patient injury.

Symbol mark for labeling

MD : Medical Device

Specifications

1. Shape

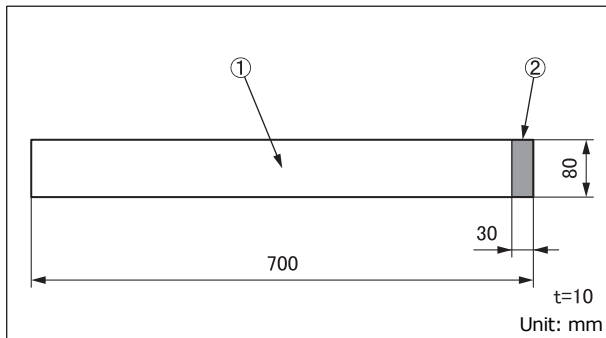

Code No.	Product Description
08-084-03	Arm strap

2. Weight

0.2 kg

3. Material

- ① Band: French pile
- ② Stopper: Velcro tape

Note: The number corresponds to those used in 1. Shape.


Intended purpose

This is an accessory for the operating table. It is a releasable band to fix the arm of patient to the Armboard.

Intended user

This product is to be used by health care professionals, including but not limited to surgeons, nurses and biomedical technicians.

Instructions for use

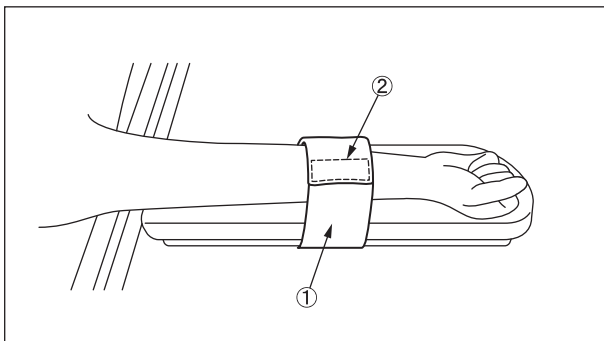

1. Place the patient's arm on the Armboard.
2. Attach the band① at a necessary position.

3. Attach the stopper② at the end of the band to the band's surface to fix the patient's arm.

Warning/Caution

1. Warning

Ensure all attaching parts are fixed. Loose condition can cause the product to come off or move, which may result in a patient injury.

2. Important caution

- 2-1. When working with an operating table, take care not to allow this product to make contact or interfere with the tabletop or with other tools and appliances used in combination with this product. Otherwise the product could break.
- 2-2. For hygiene, be sure to use sterilized drapes on the areas on this product where the patient comes into contact with it.

Storage/Life

1. Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
2. This product is a consumable. Replace it with a new product when fixing force of the fastener is weakened.

Maintenance/Inspection

<By the user>

1. Check before and after use

Check that this product is not damaged or broken before and after each use.

2. Cleaning and disinfection

Hand wash this product with a neutral household or similar detergent and let it dry naturally. This product could shrink or fade due to washing. Remove lint attached on the velcro tape as it may weaken the fixing force.

Packing

1 piece per pack

Disposal

This device must be disposed of in accordance with local regulations. Please contact your local distributor for proper disposal.

Notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<https://www.mizuho.co.jp>



EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem, The Netherlands