


Instruction for Use

Trade Name: Lower Armboard

Warning

1. When using operating table accessories, always pay close attention to the patient. Patients left in the same posture for an extended period of time are at risk for developing nerve paralysis and/or bedsores.
2. Use another armboard to hold the other hand of a patient lying in the lateral recumbent position or a similar position. Failure to do so may result in a patient injury.

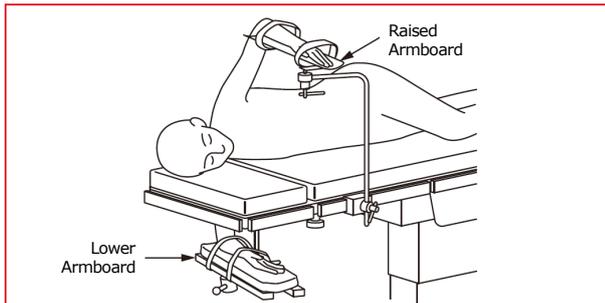

Note: The Raised Armboard needs to be purchased separately.

Contraindication/Prohibition


1. This product is only validated for use with Mizuho operating tables.
2. Do not exceed the allowable load (5 kg). Otherwise the product could break.

Symbol mark for labeling

MD : Medical Device

Specifications

1. Shape

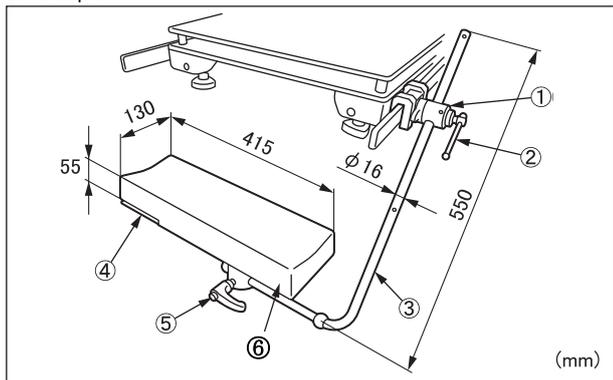

Code No.	Product Description	Weight
08-086-15	Lower Armboard with Clamp	4.4 kg
08-086-15-R1	Lower Armboard (R1)	4.4 kg
08-086-15-R2	Lower Armboard (R2)	4.4 kg
08-110-01	Clamp for Round Bar *1	0.9 kg
08-117-09	Clamp for Round Bar (R1) *2	0.9 kg
08-117-11	Clamp for Round Bar (R2) *3	0.9 kg

*1, *2 and *3: The size of mountable side rail is different.

2. Material

- ① Rail clamp: Stainless steel
- ② Fixing handle: Stainless steel
- ③ Support pole: Stainless steel
- ④ Armboard: Stainless steel
- ⑤ Adjusting lever: Nylon
- ⑥ Pad: Polyurethane

Note: The numbers correspond to those used in 1. Shape.

Intended purpose


This is an accessory for the operating table and mainly used for supporting a patient lying in the lateral recumbent position or a similar position.

Intended user


This product is to be used by health care professionals, including but not limited to surgeons, nurses and biomedical technicians.

Instructions for use

1. Attach the Armboard.

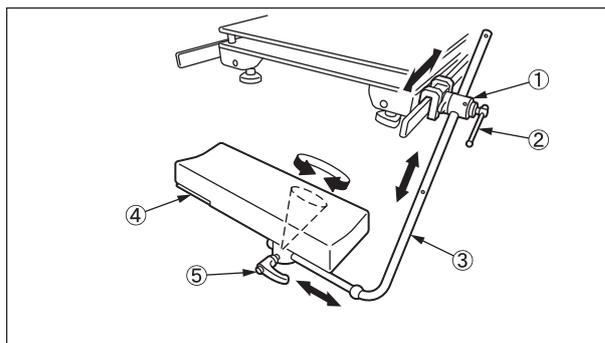

- 1-1. Attach the rail clamp① to the side rail of the operating table.
 - 1-2. Insert the support pole③ into the rail clamp① and tighten the fixing handle② to temporarily fix the support pole③.
 - 1-3. Move the support pole③ to a position appropriate for the patient's posture.
 - 1-4. After the adjustment, tighten the fixing handle② to fix the rail clamp① and the support pole③.
 - 1-5. Loosen the adjusting lever to adjust the angle of the Armboard④.
 - 1-6. After the adjustment, tighten the adjusting lever⑤ to fix the Armboard④.
- ##### 2. Move the adjusting lever

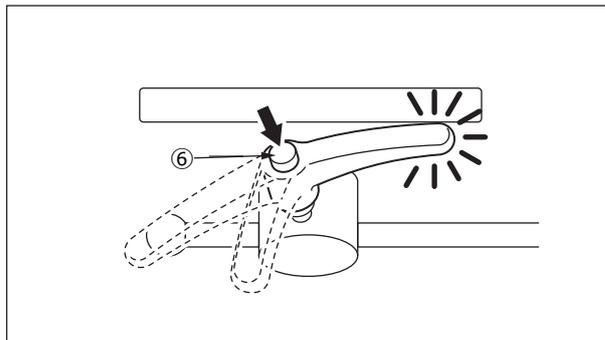

In case the adjusting lever contacting with the board and interfere with the operation, move the adjusting lever with the button⑥ pushed down.

Warning/Caution

1. Warning

Ensure all fixing handles are tight. Loose condition can cause the product to come off or move, which may result in a patient injury.

2. Important caution

- 2-1. When working with an operating table, take care not to allow this product to make contact or interfere with the tabletop or with other tools and appliances used in combination with this product. Otherwise the product could break.
- 2-2. Hold the Armboard when adjusting the adjusting lever. The Armboard will be unstable when the lever is loosen.
- 2-3. For hygiene, be sure to use sterilized drapes on the areas on this product where the patient comes into contact with it.

Storage/Life

1. Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
2. Service life of this product: 7 years
(Subject to the specified maintenance and inspection and is stored properly.)

Maintenance/Inspection

<By the user>

1. Check before and after use
Check that this product is not damaged or broken before and after each use.
2. Cleaning and disinfection
Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other materials soaked in disinfectants. In addition, use the same disinfectants as the authorized disinfectants on Mizuho's operating table described in the operating table operator's manual.
3. In case of a malfunction
When this product is broken, clearly label the device as "Broken", "Do not use", "Need repair", etc., and contact your local distributor or Mizuho.

<By agents>

Maintenance and inspection can only be carried out by certified agents of Mizuho.

Packing

1 piece per pack

Warranty

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

Disposal

This device must be disposed of in accordance with local regulations. Please contact your local distributor for proper disposal.

Notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<https://www.mizuho.co.jp>



EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem, The Netherlands