

Instruction for Use

Trade Name: Lateral Cassette Holder

Contraindication/Prohibition


This product is only validated for use with Mizuho operating tables.

Symbol mark for labeling

MD : Medical Device

Specifications

1. Shape

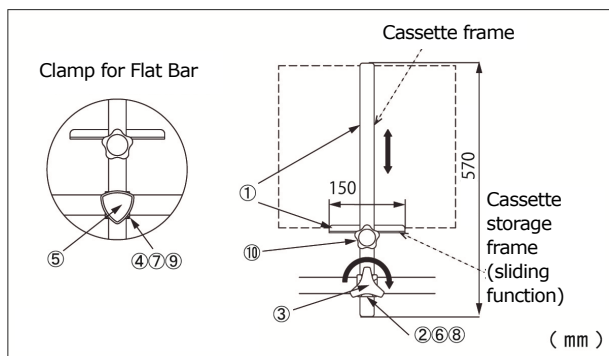

The rail clamp shown above is the Clamp for Flat Bar Ver.2.

| Code No. | Product Description | Components | Weight |
|--------------|---|------------|--------|
| 08-089-03 | Lateral Cassette Holder with Clamp *1 | ①④⑤⑩ | 1.4 kg |
| 08-089-03-NC | Lateral Cassette Holder *2 | ①⑩ | 1.2 kg |
| 08-089-05 | Lateral Cassette Holder with Clamp Ver.2 *3 | ①②③⑩ | 1.6 kg |
| 08-111-00 | Clamp for Flat Bar *4 | ④⑤ | 0.2 kg |
| 08-111-11 | Clamp for Flat Bar Ver.2 *5 | ②③ | 0.4 kg |
| 08-117-02 | Clamp for Flat Bar (R1) *6 | ⑤⑦ | 0.2 kg |
| 08-117-10 | Clamp for Flat Bar Ver.2 (R1) *7 | ③⑥ | 0.4 kg |
| 08-117-12 | Clamp for Flat Bar (R2) *8 | ⑤⑨ | 0.2 kg |
| 08-117-13 | Clamp for Flat Bar Ver.2 (R2) *9 | ③⑧ | 0.4 kg |

*1: *1 comes with *4.

*2: Select *6, *7, *8 or *9 to use *2.

*3: *3 comes with *5.

*4 to *9: The size of mountable side rail is different.

*10: The number corresponds to those used in 1. Shape.

2. Material

- ① Lateral cassette holder: Stainless steel
- ②⑥⑧ Clamp for Flat Bar Ver.2: Stainless steel
- ③ Fixing handle: Aluminum alloy, stainless steel
- ④⑦⑨ Clamp for Flat Bar: Stainless steel
- ⑤ Fixing handle: Synthetic resin, stainless steel
- ⑩ Fixing knob: Synthetic resin, stainless steel

Note: The number corresponds to those used in 1. Shape.

Intended purpose


This is an accessory for the operating table.

It is used for mounting an X-ray cassette to have an X-ray for a lateral part of a patient.

Intended user

This product is to be used by health care professionals, including but not limited to surgeons, nurses and biomedical technicians.

Instructions for use

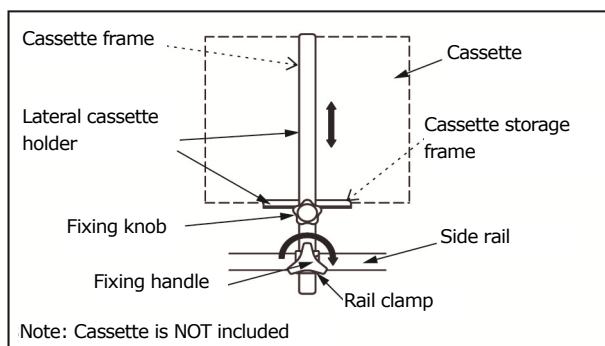

The rail clamp shown above is the Clamp for Flat Bar Ver.2.

1. Attach the rail clamp to the side rail of the operating table.
2. Insert the Lateral cassette holder into the rail clamp and tighten the fixing handle to temporarily fix the Lateral cassette holder to the side rail.
3. Attach the cassette to the Lateral cassette holder and tighten the fixing knob to fix the cassette storage frame.
4. Move and adjust this product to a position appropriate for the patient's posture.
5. After the adjustment, tighten the fixing handle to fix the rail clamp and the lateral cassette holder.

Note: The following size of cassettes can be set with the sliding function of the cassette storage frame.

254 mm x 305 mm

203 mm x 254 mm

Warning/Caution

1. Warning
 - Ensure all fixing handles are tight. Loose condition can cause the product to come off or move, which may result in a patient injury.
2. Important caution
 - 2-1. When working with an operating table, take care not to allow this product to make contact or interfere with the tabletop or with other tools and appliances used in combination with this product. Otherwise the product could break.
 - 2-2. For hygiene, be sure to use sterilized drapes on the areas on this product where the patient comes into contact with it.

Storage/Life

1. Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
2. Service life of this product: 7 years
(Subject to the specified maintenance and inspection and is stored properly.)

Maintenance/Inspection

<By the user>

1. Check before and after use
Check that this product is not damaged or broken before and after each use.
2. Cleaning and disinfection
Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other materials soaked in disinfectants. In addition, use the same disinfectants as the authorized disinfectants on Mizuho's operating table described in the operating table operator's manual.
3. In case of a malfunction
When this product is broken, clearly label the device as "Broken", "Do not use", "Need repair", etc., and contact your local dealer or Mizuho.

<By agents>

Maintenance and inspection can only be carried out by certified agents of Mizuho.

Packing

1 piece per pack

Warranty

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

Disposal

This device must be disposed of in accordance with local regulations. Please contact your local distributor for proper disposal.

Notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<https://www.mizuho.co.jp>



EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem, The Netherlands