

Instruction for Use

Trade Name: Surgical Chair

Warning

Use for intended purpose only. Incorrect use could cause breakage of this product or may result in a user injury.

Shape/ Structure

1. External view



Dimension: 472mm(W) ×450~650mm(H)

Code No.	Product Description
08-363-00	Chair SO-151

2. Weight

5 kg

3. Material

Stainless steel pipes

4. Function


Height adjustment: 200mm (450~650mm)

Intended purpose

This product is intended to be used by a surgeon during surgical procedures.

Instructions for use

Rotate the sitting part of chair and adjust height.

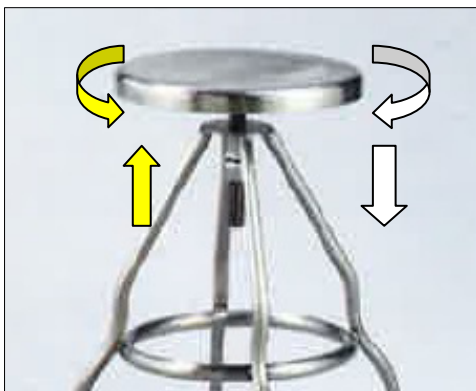

Caution

Important caution

When working with an operating table, take care not to allow this product to make contact or interfere with the table top or with other tools and appliances used in combination with this product.

Storage/Life

1. Storage

Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.

2. Service life of this product

7 years (subject to the specified maintenance and inspection and is stored properly)

Maintenance/Inspection

<By the user>

1. Check before and after use

Check that this product is not damaged or broken before and after use.

2. Cleaning and disinfection

Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other material moistened with alcohol to disinfect.

3. In case of malfunction

If you need to have this product repaired, clearly label the device as "Broken", "Need repair" etc. and contact your local dealer or Mizuho.

<By agents>

Maintenance and inspection can only be carried out by certified agents of Mizuho.

Packing

1 piece per pack

Matters related to warranty period

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation

Name and address of manufacturer



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<http://www.mizuho.co.jp>