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**EN** Sterile Drape  
For HyperEye Medical System Handy  
Instructions for use

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**REF** 08-701-40

MIZUHO Corporation  
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan

**Contact information**  
Mizuho Medical Co., Ltd.  
3-38-1, Hongo, Bunkyo-ku, Tokyo, 113-0033, Japan  
Tel:81-3-3815-7101

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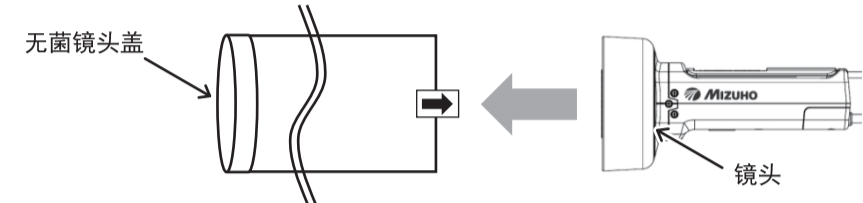
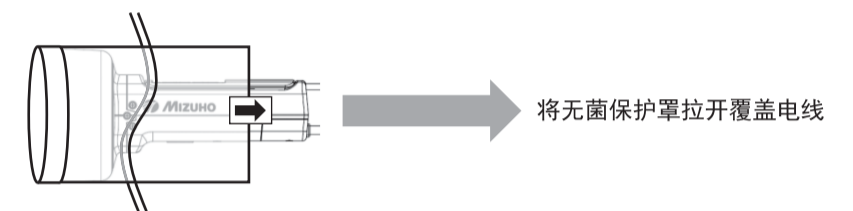
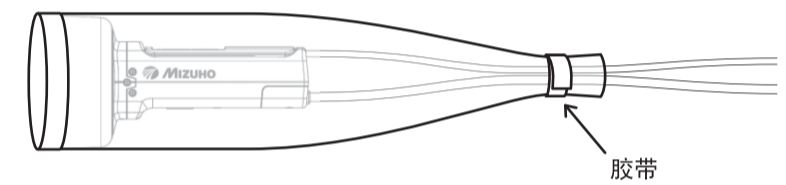
**EN**

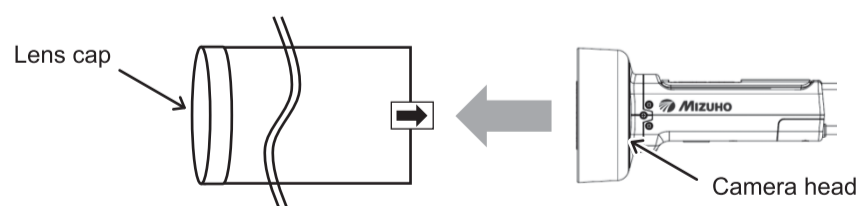
## Sterile Drape For HyperEye Medical System Handy Instructions for use

### Caution


- Store drape in a cool, dark, dry place away from direct sunlight, excessive heat, humidity and moisture.
- Check the pouch before use. Do not use if it is open or damaged, as this will compromise sterility.
- Do not use the drape if it is broken or damaged.
- Be careful not to drop the camera when applying the drape. (Camera will be broken)
- Do not hold the cable of the camera when applying the drape. (Camera will be broken)
- Always use this drape over the camera during use with a patient.
- Do not expose to or immerse in liquid. (Not waterproof)
- Do not come into contact with a patient.
- Replace the drape with a new one when a body fluid or similar adheres to the lens cap of this drape.
- Do not reuse or re-sterilise the drape. (Single use only)

### Instructions for use


1. Ensure the drape is within expiry date and is free from damage.
2. Open the pouch without touching the drape (by a person outside the sterile field).
3. Remove the drape from the pouch (by a person inside the sterile field).
4. Open the drape, put the lens cap on the camera head.

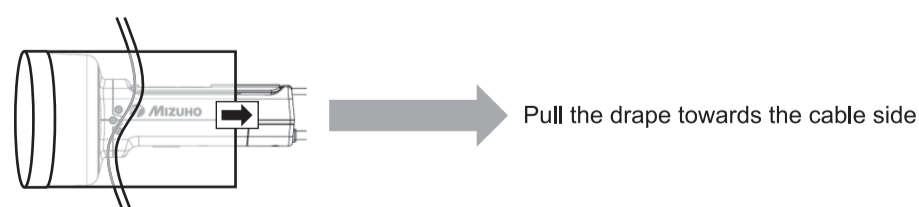5. Pull the drape towards the cable, following the direction of the arrow.  
Cover the camera and cable with the drape.

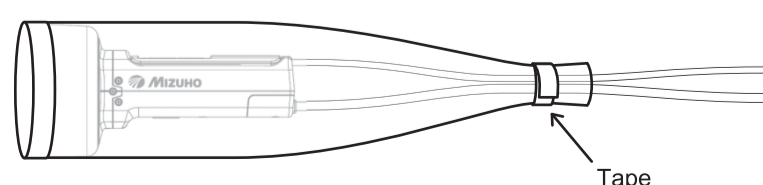6. Secure the fully extended end of the drape to the camera cables with tape, as per the following photos.

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