



Screen Protector for LCD Hand Controller Operator's Manual



Thank you for purchasing the Screen Protector for LCD Hand Controller. Make sure to read this manual in order to safely and correctly use this product. Also, carefully store this manual so that it will not be lost. If you have any questions, contact your distributor or Mizuho.

1. Introduction

This product is the Screen Protector that is used on the LCD Hand Controller.

By applying this product onto the LCD Hand Controller that you are currently using, the LCD will be protected from getting scratched.

■ Applicable Hand Controllers

- MOT-VS700 series
 - MST-7300 series
-



CAUTION

- This product is a consumable item. If it gets damaged, it is recommended that it be replaced with a new one as is needed.
 - Usage of this product does not guarantee the prevention of LCD damage.
 - Using the product in environments where there will be high temperatures and high humidities, or using it where there are chemicals may accelerate the deterioration of this product.
 - The usage period is estimated to be 1 year. It is recommended that it be periodically replaced in order for it to be safely and comfortably used.
-

2. How to Use it




CAUTION

- Before using the product, check that it does not have any defects.
- Before peeling off the Screen Protector from the backing paper, superimpose it over the LCD to confirm the application position.

■ How to apply the Screen Protector

1. Wipe the application position with a soft cloth, etc. to remove dust.
2. Pull the yellow sticker that is attached to the transparent backing paper, peel off the backing paper about 3 cm away from the Screen Protector, and align one side of the Screen Protector with the application position as per the following figure.
3. Slowly peel off the backing paper, and apply the Screen Protector onto the LCD.

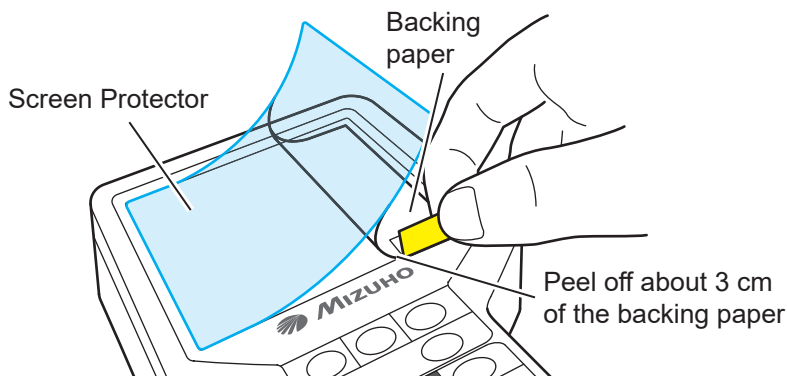

NOTE

- Remove the protective film (red tag film) that is on the LCD Hand Controller when it was purchased.
- The side that has the yellow sticker applied to it is the backing paper.
- If there are air bubbles, gently rub the surface of the screen protector with a soft cloth or your fingertips so that the air bubbles are pushed out from the center to the outside.
- To remove the Screen Protector that has been applied, apply a strong adhesive tape to one corner of the Screen Protector, and lift up that corner to cleanly peel it off.



Sales Agent



MIZUHO Corporation

3-30-13 Hongo, Bunkyo-ku
Tokyo 113-0033, Japan
<https://www.mizuho.co.jp>