

Be sure to read this instruction manual before using this product.
Keep this manual available for reference when needed.

Instruction for Use


Trade Name: Levitator Stirrup (Levitator Stirrups Cart)

Warning

Do not use for any purpose other than for storing and moving Levitator Stirrup. Misuse could cause injury or damage.

Specifications

1. Shape

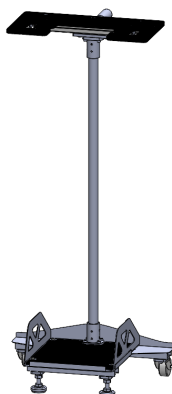

Dimension: 271 mm (L) × 416 mm (W) × 947 mm (H)

Code No.	Product Description
18-070-61	Levitator Stirrups Cart

2. Weight

8kg

3. Material

Main body: Stainless Steel, Aluminum Alloy

Caster: Resin

Intended purpose

This is a cart to store and move Levitator Stirrup.

Intended user

This product is to be used by health care professionals, including but not limited to surgeons, nurses and biomedical technicians.

Instructions for use

Store and move Levitator Stirrup as shown in the figure.



Note: The figure shows an example.

Warning/Caution

Do not move the cart when Levitator Stirrup are not stored appropriately. Failure to do so may result in Levitator Stirrup falling down and a person injury.

Storage/Life

- Do not store the device in high temperatures or in areas with high humidity where the temperature has drastic variations.
- Service life of this product: 7 years
(Subject to the specified maintenance and inspection and is stored properly.)

Maintenance / Inspection

<By the user>

- Inspection before and after use
Check that this product is not damaged or broken before and after each use.
- Cleaning and disinfection
Wipe off blood, chemicals, contaminants, and other stains with water, and clean the device with gauze or other material moistened with alcohol to disinfect.
- In case of a malfunction
When this product is broken, clearly label the device as "Broken", "Do not use", "Need repair", etc., and contact your local dealer or Mizuho.

<By agents>

Maintenance and inspection can only be carried out by certified agents of Mizuho.

Packing

1 piece per pack

Warranty

MIZUHO Corporation will repair defective parts of this product without charge for one year from the date of delivery/installment except for cases of damage caused by a third party's repair, act of nature, improper use or intentional damage. All other warranty terms and conditions are subject to regulations of MIZUHO Corporation.

Disposal

This device must be disposed of in accordance with local regulations. Please contact your local distributor for proper disposal.

Notice

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority in which the user and/or patient is established.

Manufacturer



MIZUHO Corporation
3-30-13 Hongo, Bunkyo-ku, Tokyo 113-0033, Japan
<http://www.mizuho.co.jp>